

Normlgel® Ag




ANTIMICROBIAL SILVER WOUND GEL

Normlgel® Ag is an antimicrobial hydrogel that is effective in assisting the debridement and desloughing process in dry necrotic wounds, while maintaining a moist wound environment for optimal wound healing. **Normlgel® Ag** contains an antimicrobial silver compound that is an effective barrier to bacterial penetration by inhibiting the growth of broad spectrum of microorganisms.¹




For wounds with dry to moderate exudate
Provides a three day antimicrobial barrier and the moisture donating benefits of a hydrogel.²


How to use Normlgel® Ag

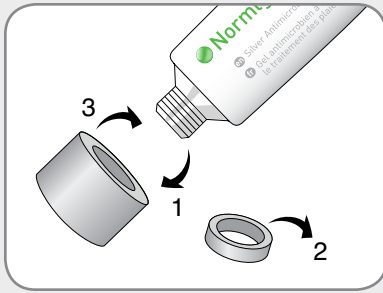

Unscrew and remove cap and distance ring. Re-apply the cap to the tube and twist it firmly back on again to break the seal of the tube.

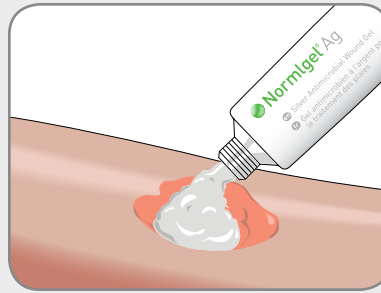

Cleanse or gently flush the wound with normal saline solution or an appropriate wound cleanser. Apply Normlgel® Ag onto the wound bed (approximately 0.2 inches thick). Ensure that Normlgel® Ag does not cover intact skin.

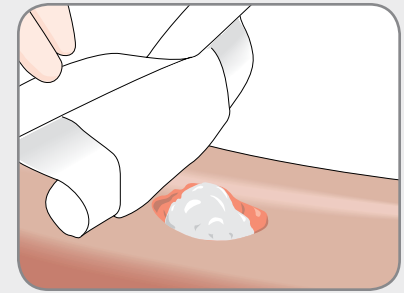

Cover the gel with an appropriate moisture retentive secondary dressing such as Mepilex® Border or Mepilex® Border Lite depending on the exudate level.

- **When to change the dressing:**
Normlgel® Ag may be left in place for up to 3 days. Dressing changes may be required more frequently dependent upon wound conditions and the levels of exudate.

- **How to change the dressing:**
Carefully remove the secondary dressing. Cleanse or gently flush the wound with normal saline solution to remove necrotic debris.

Areas of use

Normlgel® Ag is indicated under the medical supervision of a healthcare professional for the management of dry to moderate exuding partial and full thickness wounds such as:

- diabetic ulcers
- pressure ulcers
- leg ulcers
- graft and donor sites
- post-operative surgical wounds
- trauma wounds
- superficial and partial thickness burns
- abrasions and lacerations

Wear time: Up to 3 days

Key benefits of Normlgel® Ag

- Adds moisture and maintains a moist wound environment
- Assists debridement and desloughing in dry and/or necrotic wounds
- Effective barrier to bacterial penetration
- Tissue-friendly
- Easy to use
- Ideal viscosity makes gel easy to apply and stay in place

Contraindication: Do not use on patients with a known sensitivity to silver or propylene glycol.



Normlgel® Ag Ordering Information†				
Product Code	Size	Pieces/ box	Pieces/ case	HCPCS
350450	1.5 oz (45 g)	10	30	A6248

† Normlgel® Ag is unsterile and packed in tubes

1. Normlgel® Ag-Comparison Report (properties SilverGel SilvaSorb) P2033 PD-417187
2. Study summary, Silver Gel-Normlgel® Ag PD-417194



5550 Peachtree Parkway, Suite 500, Norcross, GA 30092 | 800.882.4582 | www.molnycke.us | www.mhcwoundcare.com

The Mölnlycke Health Care, Mepilex, and Normlgel names and respective logos are registered trademarks of Mölnlycke Health Care AB. Distributed by Mölnlycke Health Care US, LLC, Norcross, Georgia 30092. © 2011. Mölnlycke Health Care AB. 1.800.882.4582.

MHC-2011-0392 - 270369 09.11