

## **1589 Wound Gel Normlgel Instructions**

### **Instructions:**

1. Remove secondary dressing and discard in appropriate way.
2. If necessary, gently clean/flush the wound with normal saline solution.
3. if clinically indicated, protect the surrounding skin with a water resistant cream/ointment.
4. Open foil tube. Refer to drawing
5. Apply a liberal amount of gel to cover the entire wound surface but not the surrounding healthy skin, and cover with a moisture retentive, absorbent dressing.
6. In deep wounds, pack loose with a suitable absorbent material.
7. Change Normlgel every 48 hours, or as indicated by the clinical condition of the wound.

